



CANCELLO-PURE™

Evans and Cotton
Bone Wedges



WRIGHT.

CANCELLO-PURE™ Wedges



Evans Wedge

Cotton Wedge

STRONG alternative to allograft.

"The CANCELLO-PURE™ Wedge is an attractive alternative to allograft. It's strong, safe, and readily available off-the-shelf. The new sizes are custom-fit and easy to use for Cotton and Evans osteotomies."

Hodges Davis, MD
OrthoCarolina - Charlotte, NC

SAFE


BIOCLEANSE® treated bovine bone renders a sterile implant that is available off-the-shelf

SPECIFIC


Engineered for Evans and Cotton osteotomies

SUCCESSFUL

Incorporates into host bone

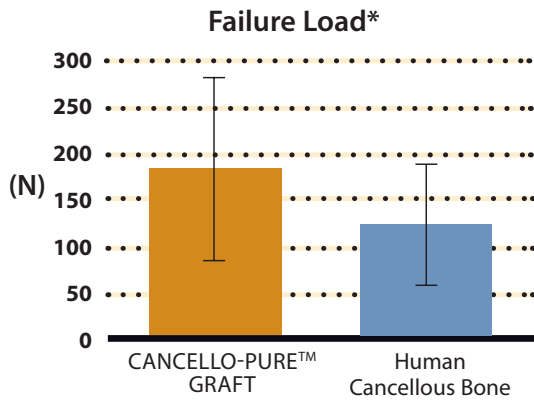

**The relative mechanical performance of CANCELLO-PURE™ Wedge was compared to similarly processed human cancellous bone under axial compression to determine failure load. Six samples were provided for each group that were 6mm in diameter x 12mm in length. After hydration, each sample was tested under static axial compression on an MTS machine at a rate of 1.3mm/min until failure. Failure was considered an occurrence of either axial deformation of 1.5mm, or a decrease in sustained load of 100N. The CANCELLO-PURE™ Wedge demonstrated a stronger average compressive strength than allograft cancellous bone, although the results were not statistically significant.*

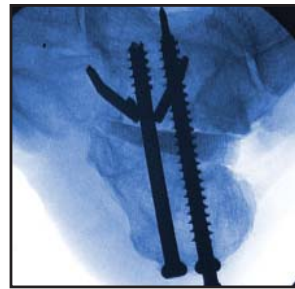


Intra-Op



Post-Op, 10 Weeks

Case Example - Revision of malunited triple arthrodesis
Images courtesy of Hodges Davis, MD

ORDERING Information

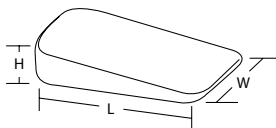

PART NO.	DESCRIPTION	HEIGHT x LENGTH x WIDTH (MM)
46660012	CANCELLO-PURE™ EVANS Bone Wedge	12 x 38 x 22
46620006	CANCELLO-PURE™ COTTON Bone Wedge	6 x 24 x 14
96001072	CANCELLO-PURE™ Bone Wedge	10 x 50 x 40

Sterilized through the BioCleanse® process. Stored hydrated at room temperature for off-the-shelf use.



Wright Medical Technology, Inc.
5677 Airline Road
Arlington, TN 38002
901.867.9971 phone
800.238.7188 toll-free
www.wmt.com

Wright Medical Europe SA
Rue Pasteur BP 222
83089 Toulon Cedex 9
France
011.33.49.408.7788 phone
www.wmt-emea.com

Refer to Instructions For Use included with the product prior to use.

For non Load-bearing use only.
Use with appropriate primary fixation

™Trademarks and ®Registered marks of Wright Medical Technology, Inc. Product processed by Regeneration Technologies, Inc. for Wright Medical Technology, Inc. BIOCLEANSE® is a Registered trademark of Regeneration Technologies, Inc. Covered by one or more of the following US patents: 6,482,584; 6,045,554; 6,482,584; 6,613,278; 6,652,818; 6,497,726; 6,699,252. Patents pending.
©2007 Wright Medical Technology, Inc. All Rights Reserved.